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MANUAL *Erection System*®

Important:

Please read these instructions for use before using the erection aid. Keep the instructions for use! Please watch the animation on the enclosed DVD.

Instructions for Use

When the penis is stiff, this is called an erection. If a man has problems getting an erection, we speak of erectile dysfunction. The **MANUAL Erection System**® is a device that helps you to get your penis stiff. Therefore in this booklet the **MANUAL Erection System**® is also referred to as an erection aid.

What are the causes of erectile dysfunction?

A variety of causes can lead to erectile dysfunction. The most common ones include diabetes, thyroid disease, liver or kidney disease, Treatment of hypertension, cardiovascular diseases. Prostate surgery often also leads to erectile dysfunction. Depression or exhaustion can also lead to erectile dysfunction.

Sometimes erectile dysfunction is a side effect of medication, e.g. drugs against high blood pressure. Smoking or drinking too much alcohol is another possible cause.

The MANUAL *Erection System*® comprises the following parts:

- 1 pump head [1]
- 1 transparent cylinder [2]
- 1 large grey ring loader [3]
- 1 small grey ring loader [4]
- 1 grey cone [5]

- 4 constriction rings in differing sizes [6]
- 1 tube of lubricant
- 1 set of instructions for use with DVD
- 1 bag

Please note the shelf life of the included constriction rings and the lubricant.

How to prepare the application?

All product-related illustrations can be found on the cover flap of this booklet.

- Place all parts at hand. (Fig. [1])
- Press the pump head onto the larger opening of the cylinder, as shown in Figure [2].
- Plug the large ring loader onto the other side of the cylinder. (Fig. [3]).
- Insert the small ring loader into the large ring loader.
- Insert the cone into the small ring loader. (Fig. [3])
- Apply sufficient lubricant onto the entire cone. (Fig. [5])
- First, place the largest constriction ring (20 mm) on the top of the cone. Slide the constriction ring over the cone onto the small ring loader. Now take off the cone. (Fig. [6])
- Now the erection aid is ready for use.

How to use the erection aid?

The erection aid is exclusively for your personal use. It may not be passed on to third parties, nor be used by them.

- Remove the pubic hair directly on the penis and around the root of the penis.

- The device must fit the skin tightly.
- Apply some lubricant to the edge of the ring loader. (Fig. 7)
- Introduce the penis through the ring loader into the cylinder. (Fig. 8)
- Press the device tightly to your body.
- Pull the blue lever on the pump head. Thus, you pump air out of the cylinder. (Fig. 8)

Do you see a good erection in the cylinder?

- Then slide the constriction ring from the ring loader onto the erect penis .
- Slide the constriction ring quickly and as far as possible down to the base of the penis. (Fig. 9)
- The “air” button on the pump head is marked with a dot. (Fig. 10)
- When you press the “air” button, the cylinder is bled (i.e. air is let in, reducing the vacuum).
- Thus air enters the cylinder, and you can easily detach the erection aid from the body.
- The constriction ring on the penis causes the penis to remain rigid. (Fig. 11)
- Now you can have sex.

Important: Never wear the constriction ring on the penis for more than 30 minutes. Do not fall asleep with the constriction ring on the penis!

After sex:

- Briefly pull the constriction ring apart on the flaps.
- Wait until the blood has flown back from the penis into the body.
- Then pull the constriction ring off the penis. When you use the erection aid, you will have no visible ejaculation upon orgasm (climax). (Fig. 12)
- The constriction ring retains the semen (ejaculate) in the penis.
- Once you remove the constriction ring, the semen will flow out of the penis.
- The temporary retention of the semen in the penis is not harmful.

- After using the erection aid, clean it as described in Section “How to clean the erection aid?”
- Dry the parts.
- Put all parts back into the bag and keep the erection aid out of the reach of children.

How does the erection aid work?

The pump head sucks the air around the penis out of the cylinder. This creates a negative pressure (vacuum). This causes the blood vessels in the penis to fill with blood, and the penis stiffens. As the cylinder is transparent, you can see whether the erection is hard enough. In most cases, pumping for 1 to 2 minutes will be sufficient to produce an erection.

Application notes

Application of this erection aid is easy. Nevertheless, it is better to try the erection aid initially alone and unhurried. Then you can show the erection aid to your partner. Once you both know how to use the erection aid correctly, you can enjoy sex again. Even at an advanced age, in illness or physical limitation.

How to select the proper constriction ring?

To keep your erection as long as possible, you need to choose the proper constriction ring. If the constriction ring is too large, your penis will not remain stiff for long. If the constriction ring is too tight, it may cause pain. In case of pain, take off the constriction ring and try a larger constriction ring. If you use the erection aid for the first time, it is best to start with the largest constriction ring. If the constriction ring is too large, try a smaller one, until you find the right size for yourself.

The erection aid includes 2 ring loaders.

The smaller one is slightly narrower, the larger a bit wider. If the smaller ring loader is too narrow, just use the larger one. The two ring loaders are suit-

able **for all constriction rings**. Thus, it does not matter which constriction ring size you are using. If both ring loaders are too narrow, you can slip the constriction ring directly onto the cylinder. The erection aid also includes the **cone**. It helps you to slip the constriction ring more easily onto the ring loaders. Alternatively, you can slip the constriction ring manually onto the ring loaders without using the cone. Use only the supplied constriction rings. From time to time, the constriction rings need to be replaced. They can be ordered from **service@medintim.de**

Never use this erection aid if you have drunk alcohol or have taken recreational drugs. Do not continue using the erection aid if it does not work properly. Do not use the erection aid either if it is visibly damaged. In both cases, please send the erection aid directly to the manufacturer, **KESSEL medintim GmbH**.

During the warranty period, repairs are free. If the device is not working or damaged because you have operated it incorrectly, you will always have to pay for the repairs.

If you do not live in Germany, please contact the dealer from whom you bought the erection aid. Use only lubricants that can easily be washed off with water.

Do you have any questions, or do you need help?

Feel free to call this number:

+49 (0) 6105 20 37 20

Or send an email to:

service@medintim.de

What side effects can the erection aid have?

Do you suffer from cardiovascular disease?

Then talk to your doctor or health care professional before using the erection aid for the first time.

The following side effects are possible:

You can get a bruise (haematoma) at the base or at the top of the penis.

Sometimes there may even be slight tissue injury, when the penis was not erect for a long time.

However, that is not dangerous.

All parts have been tested. However, contact allergies cannot be excluded.

When may the erection aid not be used?

Do not use the erection aid if you have any of the following conditions:

- Diseases that affect the red blood cells, e.g. sickle cell anaemia
 - Blood cancer (leukaemia)
 - Bone marrow tumours
 - Diseases that affect blood clotting (coagulation)
 - Diseases that cause bleeding or prolonged erections
 - Other diseases in which sex is not recommendable
- If you are not sure whether you can apply the erection aid without any problem for your health, ask your doctor or health care professional.

How to clean the erection aid?

Before cleaning, take the pump head off the erection aid. All parts can be easily cleaned with warm water and mild soap. Do not use aggressive detergents. Clean the erection aid after each use. Store the erection aid in the included bag in order to avoid contamination.

How to recycle the erection aid correctly?

All parts can be disposed of into the household waste.

Warranty

Almost all parts are warranted for 2 years, as prescribed by applicable law.

Wear parts, such as constriction rings, are not covered by warranty. They must be replaced from time to time.

The warranty does not apply if you damage the erection aid or use it incorrectly.

The pump head is sealed inside and may be opened only by the manufacturer or an authorised service partner. You should send in the erection aid, if possible, every 3 years for technical inspection.

What material are the parts made of?

- Cone: Plastic (acrylonitrile butadiene styrene, ABS)
- Small and large ring loader: Plastic (acrylonitrile butadiene styrene, ABS)
- Cylinder: Plastic (polycarbonate)
- Constriction rings: Biocompatible Thermoplastic Elastomer (TPE)
- Bag: Nylon®

You can reorder all the parts and the lubricant. To get the current reordering list, please contact **service@medintim.de** or **info@savantini.com** for UK (+44 0142 888 785) or your dealer.

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FAQ MES

Medical questions on use of the MANUAL Erection System®

Can any man use the MANUAL Erection System®?

Yes, usually. It will help any man to achieve an erection again and maintain it, whether he is young or old, healthy or ill.

Can I use the MANUAL Erection System® if I have diabetes?

Yes, it is especially useful for diabetic patients, as other methods are often not very helpful.

Can I use the MANUAL Erection System® if I have high blood pressure?

Yes, the vacuum pump is very effective in helping men who are taking medications for high blood pressure to achieve an erection. In fact the cost of the unit is usually reimbursed by the statutory health insurance agency.

Can I use the MANUAL Erection System® if I am taking anticoagulants (blood thinners) such as warfarin?

Generally, yes. Using a vacuum pump is not a health risk for users of anticoagulants (blood-thinning medications). However, there could be (strong) bruising, which might make continued use of a vacuum erection appliance unwise. Serious or permanent injury is not possible.

Can I use the MANUAL Erection System® after prostate surgery?

Yes, in our experience the MANUAL Erection System® is in this case even better than the battery-operated ACTIVE Erection System^{NT}®.

Can I use the MANUAL Erection System® after radiotherapy (prostate)?

Yes, use of a vacuum pump such as the MANUAL Erection System® is useful and effective after radiotherapy of the prostate.

Can I use the MANUAL Erection System® after a radical prostatectomy?

Generally, yes. In our experience, the MANUAL Erection System® is better in this case. This pump creates a vacuum (sub pressure and suction) rather more quickly and effectively. We would be happy to advise you.

Since my prostate was removed, I sometimes find it difficult to hold my urine in. Can I still use a vacuum pump?

Yes, we recommend the MANUAL Erection System® in this case, because this unit is easier to clean if any urine gets into the cylinder while using the pump.

Is it there an increased risk of thrombosis (blood clots) if I use the vacuum pump?

No, treatment with the vacuum pump does not affect your risk of thrombosis (blood clots).

I am over 70 – can I still use the MANUAL Erection System®?

Yes, of course. We have many customers who are over 70 and even in their mid 80's. In fact, our oldest customer, who we spoke to personally, was 92 years.

Sexual activity has no age limits. Vacuum pumps are a simple and practical aid.

Can I use the MANUAL Erection System® if I have a curved penis (Peyronie's disease)?

Yes, as long as the curvature of the penis does not prevent you from inserting it into the cylinder.

We have received several positive comments from patients, who said that regular use of the vacuum pump (daily two times sub pressure around the

penis and a better blood flow) reduced the curvature of the penis. Studies in the US show: This kind of a "training of cavernous tissue" reduces the angle of the curved penis and avoid to 47% any surgery as a treatment.

Can I use the MANUAL Erection System® if I have mild Parkinson's disease?

Yes, you can use a vacuum pump if you have Parkinson's disease.

Can I use a vacuum pump if I have MS (multiple sclerosis)?

This neurological disorder often results in weak or inadequate erections, lack of orgasm or failure to ejaculate. Use of the MANUAL Erection System® can help to solve these problems. A vacuum pump can be used.

With which diseases can the MANUAL Erection System® (vacuum pump) not be used?

In sickle cell anaemia, leukaemia, bone marrow cancer or other diseases that alter blood clotting; this could result in bleeding and prolonged erections. If there are any other medical reasons not to engage in sexual activity, the appliance should not be used.

I have a bruise on my penis after using the vacuum pump. Is this dangerous?

No, that sometimes happens – particularly with diabetes. The bruise will disappear in a few days.

Can my penis get bigger when I use the vacuum pump?

Yes, this is possible, because the vacuum increases blood flow to the penis and expands the erectile tissue.

How often can I use the MANUAL Erection System®? Is it possible to use it several times a day?

You can use the unit as often as you like. There is no risk of injury.

Can I generate too much suction or injure myself by using the vacuum pump?

No, the strength of the vacuum (sub pressure) cannot exceed a set value and you cannot injure yourself.

Sometimes there is a feeling of tension when I use the unit. I find this unpleasant.

Select a lower level of suction and perhaps let a little air in (blue inlet button at the handle). With the vacuum pump, the quality of an erection is better if it is developed slowly.

If I use the MANUAL Erection System®, will I have normal sex with an orgasm?

Yes, with the artificial erection you can have normal sex or masturbate for up to half an hour (30 min.). You can still have an orgasm as before.

Will I still ejaculate if I use the vacuum pump?

You can ejaculate, but the semen and the prostate fluids is held in by the constriction ring. After taking off the constriction ring, the ejaculate will flow out. A visible "spurt" at orgasm is not possible.

Isn't it a problem if the semen can't flow out at orgasm?

No, the vessels can store it and release it later.

Can I still father a child if I use the MANUAL Erection System®?

If you want to have a child, you can use a special ring, the ACTIVE LOOP RING (ALR). This ring can be opened after orgasm with a small tab, so that the semen can flow out while the penis is still in the vagina.

If you do not want to father a child, the constriction rings are not a safe means of contraception, and a safe form of contraception should be used.

Is it possible that my penis has got bigger than before by using the vacuum pump?

You may have this impression because you didn't

have a proper erection for a long time. Using the vacuum pump increases the blood supply to the penis. The penis can then enlarge more than it could with a poorer blood supply.

I use a method in which I inject a drug into the shaft of the penis. Can I still use the MANUAL Erection System®?

Yes, there are quite a number of users of this medicinal method for treating erectile dysfunction. They start by developing an erection with the MANUAL Erection System®, and then inject the drug into the erect penis, which is now well supplied with blood. In this way the drug is distributed better through the erectile tissue.

My partner thinks that my penis feels colder when we use the pump.

Yes, this is possible, because the constriction ring reduces blood flow.

Solution: you can apply a warm gel beforehand, or briefly wrap hot towels or a hot water bottle around the penis.

Practical questions on use of the system

I can't put the parts of the MANUAL Erection System® together.

First have another look at the steps in the instruction manual. Have you watched the DVD that came with the unit? If it does not work, we can help you further on the phone at + 49 6105 203720 or send us an e-mail at: service@medintim.de

The unit does not work. What should I do?

First, check to see whether all parts are properly fit together. Did you shave the pubic hair at the shaft of the penis, where the cylinder meets your body?

What should I do if the motor is running, but the unit is not producing a vacuum (not sucking).

You can test to see if the motor is generating enough suction. Hold the motor with its cylinder on a flat part of your body, e.g. your thigh or your abdomen. If the cylinder sucks onto your skin and makes a "plop" sound when you take it off, the unit is working.

Try it again on the penis and make sure that there is a tight seal between the cylinder and the shaft of the penis. A little lubricant (e.g., Animé or any other water-based lubricant) helps to make a good seal between the cylinder and the shaft.

What can I do if I pumped several times, but there is no vacuum or erection visible in the cylinder?

Send us the MANUAL pump head in a prepaid package and we will check it. You have a two year warranty. Don't forget to give the address of the sender!

My erection does not last long, even with the constriction ring.

Try using a smaller, tighter constriction ring.

I have the hand pump head of the MANUAL Erection System®. Can I use the battery-operated pump head from the ACTIVE Erection System^{NT}® on my unit?

Yes, that will work. The devices are compatible. Each pump head fits the cylinder of the other system.

My erection doesn't last long, even with the smaller constriction ring.

It is possible to use two constriction rings together. There are also specially made elongated guide sleeves, which you can find on the list of spare parts. We would be happy to advise you.

The erection is still good as long as I'm pumping with the cylinder, but as soon as I put the constriction ring on the erection gets weaker. Then I can't really have sex.

There is a little trick that will help you to maintain a really firm erection with "re-pumping". After slipping the constriction ring on, you use the pump again.

Remove the grey guide sleeves from the cylinder. Now put the just the pump with the cylinder over your penis, which already has the constriction ring in place. Press the cylinder firmly against the shaft and produce a vacuum around the penis again. Blood can now flow into the penis, but it can't flow out again. After you take the cylinder off, the erection will now be really firm and at right angles to your body.

I use the large guide sleeve, but it is difficult to pull the glans out of the cylinder through the large guide sleeve when the penis is erect.

Don't use the guide sleeve at all, just stretch the constriction ring directly onto the cylinder. Then it should work.

When my penis is erect, the glans is up against the pump head. Is there a longer or larger cylinder?

Yes, there is cylinder that is about 5 cm longer and 1 cm larger in its diameter. You may find in our parts list. We would be happy to advise you.

My penis completely fills the cylinder when it's erect, and it's difficult to pull it out. Is there a larger cylinder?

Yes, there is our "Special Plus" system. This has a cylinder of 25 cm length and 6 cm inner diameter. This cylinder fits onto the pump head with an adapter. We would be happy to advise you.

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