Instructions for use

Autopen®

Cartridge	Increments	Model No
3.0 ml	2–42	AN3800
3.0 ml	1–21	AN3810
3.0 ml	2–42	AN4000
3.0 ml	1–21	AN4010
3.0 ml	2–42	AN4100
3.0 ml	1–21	AN4110

Autopen is an easy-to-use, multi-dose insulin injection pen, which provides a convenient and discreet method of delivering a preset dose of insulin from a 3.0 ml cartridge produced by Eli Lilly or CP Pharmaceuticals.

The 1 and 2 Unit 3.0 ml Autopen types (listed above) must not be used with insulin cartridges manufactured by Novo Nordisk or Aventis.

COMPONENTS

- Dose selector
- Device body
- Release button
- Internal plunger
- Cartridge
- Device cartridge holder
- Needle & protective caps
- Device cap
- Release button extension – simply clips onto release button for increased grip
- Dose selector adapter – fits onto dose selector for easier dialling

Unifine® Pentips®

PEN NEEDLES

Sterility is guaranteed if the needle seal is undamaged.

STERILE EO

PLEASE FOLLOW THESE STEP-BY-STEP INSTRUCTIONS CAREFULLY BEFORE USING THE AUTOPEN.

IMPORTANT SAFETY PRECAUTIONS

Failure to follow these instructions may result in an inaccurate dose.

- Always check before each injection that the Autopen contains the correct insulin type to be injected.
- Always expel any air, with the needle pointing upwards, before each injection.
- The needle should be removed and safely disposed of immediately after each injection. If the needle is not removed, temperature changes may result in some liquid being expelled from the cartridge which might cause a change in the insulin concentration.
- Your healthcare professional has prescribed the type of insulin that he/she believes is best for you. Do not use any other insulin except on the advice and direction of your healthcare professional.
- Read and follow the manufacturers instructions provided with your insulin cartridges.
- 3.0 ml cartridges from Novo Nordisk and Aventis should not be used with any of the Autopens shown in these User Instructions.
- Consult your healthcare professional should you encounter abnormal blood glucose levels.

GETTING READY TO USE THE AUTOPEN

1. Remove the device cap and unscrew the device body from the cartridge holder.
2. Check to ensure that the internal plunger is fully retracted (as in figure 2). If the internal plunger is extended, please refer to step 11.

***If using longer acting NPH (Basal) insulin you will need to resuspend the insulin by turning the cartridge up and down and rolling it in the palm as per the cartridge manufacturers instructions.**

3. Load the insulin cartridge into the cartridge holder with the metal cap in first.
4. Screw the cartridge holder and body firmly together.

PRIMING AND INJECTING

5. Attach the needle by firmly screwing it onto the threads at the end of the device cartridge holder. Remove the outer protective cap and needle cap from the needle.


***If the cartridge contains longer acting NPH (Basal) insulin, turn the pen up and down at least 10 times as per cartridge instructions. Then attach the needle to the device cartridge holder.**

6. It is important to "Prime" the Autopen before use to remove any air that may be inside the needle. Dial 2 units on the dose selector.

7. Hold the Autopen with the needle pointing up. Push and hold the release button towards the needle until the arrow ► on the pen body points to the starting line ■ on the dose selector. Repeat steps 6 & 7 until a drop of insulin appears at the needle tip.

The Autopen is now ready for use.


8. Make sure the arrow ► on the pen body points to the starting line ■ on the dose selector.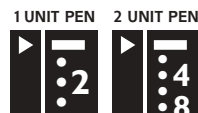

The **1 Unit 3.0 ml Autopen** dose selector measures up to 21 units of insulin in 1 unit increments.

The **2 Unit 3.0 ml Autopen** dose selector measures up to 42 units of insulin in 2 unit increments.

Now dial up the number of units required.

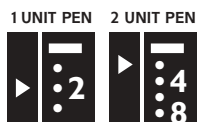Make sure before injecting that the arrow ► points towards the correct number of units on the dose selector.

For example, the diagram here shows the correct position for 2 units of insulin delivery.

9. Insert the needle (use the injection technique recommended by your healthcare professional), push and hold down the release button towards the needle until the starting line on the dose selector returns to the arrow on the body ► ■.

Count to 10 and remove the needle from the skin. Reattach the white outer protective needle cap and remove the needle. Replace the device cap onto the Autopen.

Dispose of used needles carefully in accordance with the advice given by your healthcare professional or local regulations.

If the dose selector stops before the starting line returns to the arrow, this indicates that you have not received a full dose. The dose selector will show the number of units that must be injected to complete the insulin dose.

Before making subsequent injections always make sure that a NEW pen needle is used, the pen is primed and a drop of insulin is at the needle tip. If necessary repeat steps 6 & 7.

CHANGING THE CARTRIDGE

10. To change the cartridge, unscrew and remove the needle. Unscrew the device cartridge holder from the device body and remove the cartridge.

11. Rewind the internal plunger by holding the white lower part of the device body between the thumb and finger. Turn the body until the internal plunger is fully retracted. Then return to step 2 to load the next cartridge.

Autopen should be used only after consultation with your healthcare professional.

STORAGE

- The Autopen should always be stored or carried with the needle removed, and the device cap on.
- Cartridges can be carried in the Autopen or as spares according to the cartridge instructions.
- Please store your insulin at the temperature recommended by the insulin manufacturer.
- To clean the Autopen, wipe with a damp cloth.
- **Do not immerse it in water.**

Please check contents of the package as listed on the outside of the box.

The Autopen® has been thoroughly tested and meets the dose accuracy requirements of ISO 11608-1.

Autopen®, Unifine® and Pentips® are registered trademarks of Owen Mumford. Novo Nordisk® is a registered trademark of Novo Nordisk A/S. Aventis™ is a trademark of the Aventis group of companies.

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Owen Mumford Ltd