







Hersteller: / Κατασκευάζεται από την: /
Fabricado por: / Fabriqué par: /
Fabbicante: / Gefabriceerd door: /
Fabricado por: / Fremstillet af: /
Valmistaja: / Proizvođač: / Produsert av: /
Производитель: / Tillverkad av: /
Izdeluje: / Manufactured by:

HTL-STREFA S.A.
95-035 Ozorków,
7 Adamówek St., Poland

Glucoject®
Dual PLUS

A.MENARINI
diagnostics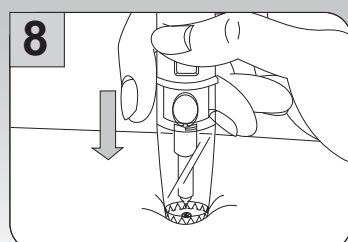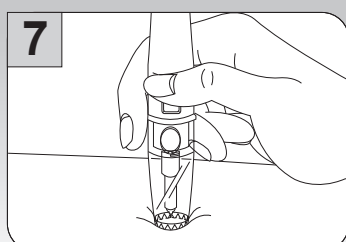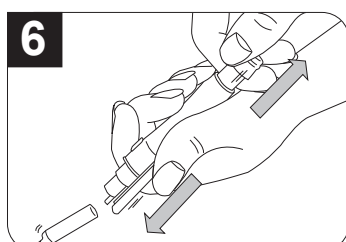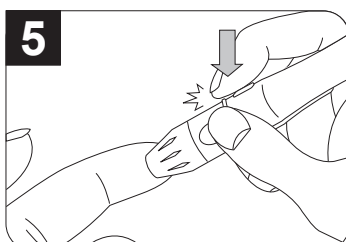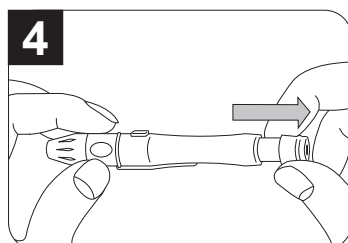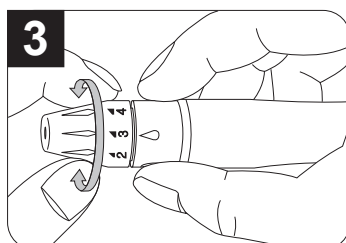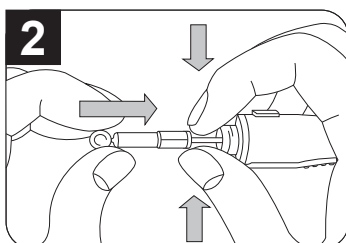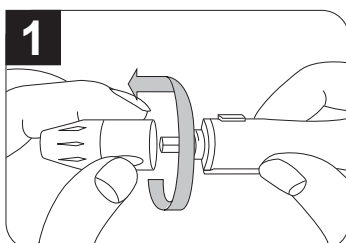

DEUTSCH

Glucoject® Dual PLUS STECHHILFE

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WARNUNGEN

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ENGLISH

Glucoject® Dual PLUS LANCING DEVICE

INSTRUCTIONS FOR USE

WARNINGS

- Must NEVER be used on more than one person.
- Use a new lancet for each puncture and do not store lancets in device.

HOW TO USE Glucoject® Dual PLUS

Wash your hands or alternative test site with warm water and soap. Rinse and dry thoroughly before the puncture. Please read the following steps with corresponding illustrations to understand how to use the Glucoject® Dual PLUS.

1 Unscrew the tip of the lancing device by turning it counter-clockwise.

2 Insert a new sterile lancet into lancet holder and push down until fully inserted.

3 Set your preferred depth of penetration by rotating the tip. 1-2 for soft or thin skin, 3 for average skin, 4-5 for thick or calloused skin.

4 Pull out the sliding barrel to load the lancing device.

5 Hold the device firmly against the finger and press the release button.

6 To remove the used lancet push the lancet ejector forward with the thumb and simultaneously pull out the sliding barrel.

USING THE CLEAR TIP (AST)

7 Place the device on the skin in the chosen area, press and hold continuously for a few seconds, then push the release button to take a sample.

8 Watch through the clear tip until a sufficient blood sample is taken. If there is not enough blood, gently massage the area until a sufficient sample has been collected.

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Med-nett A/S
A. Menarini Diagnostics Srl, Italien filial (Sverige)
A. MENARINI DIAGNOSTICS LTD
A. MENARINI DIAGNOSTICS S.r.l.